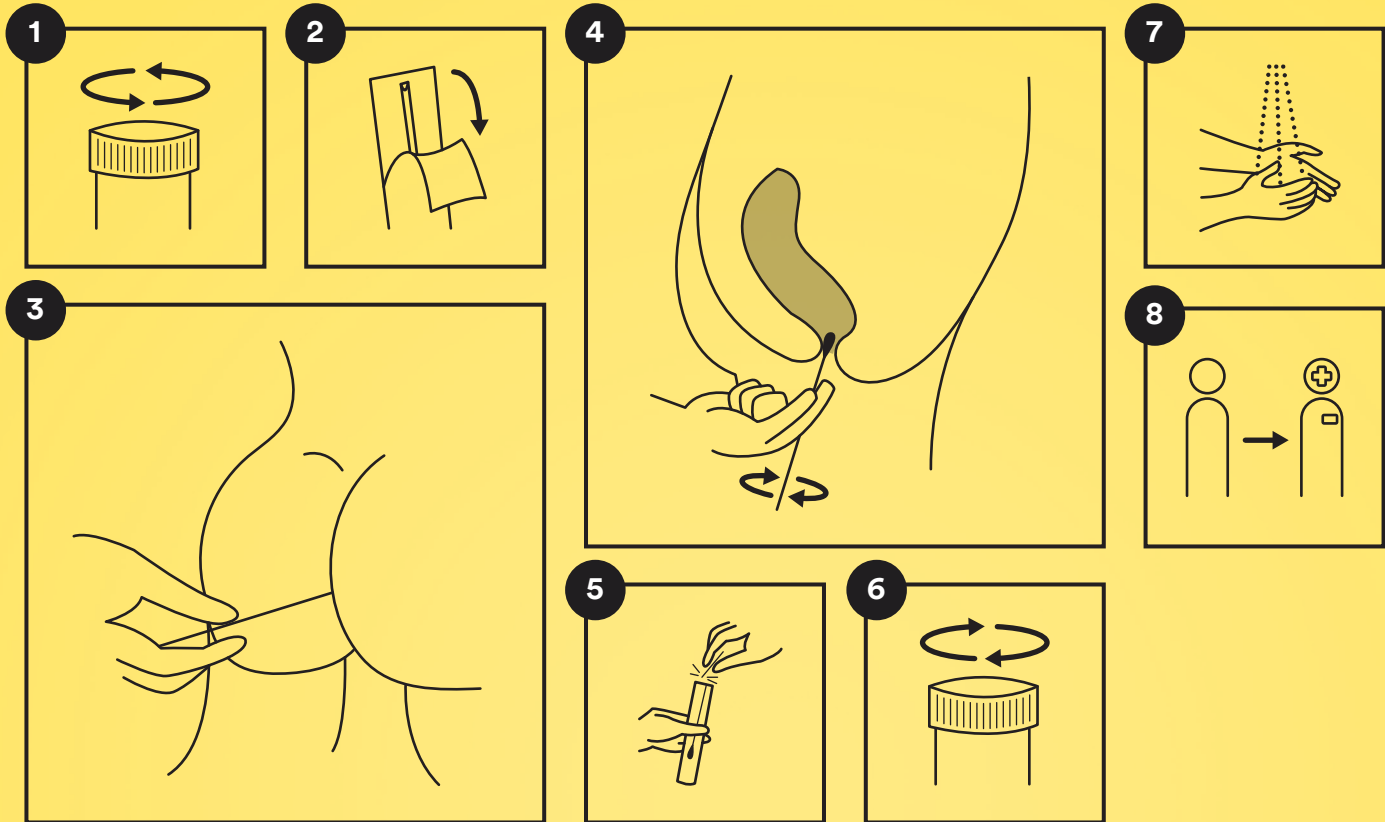
Abstrich Anus Frottis Anus

Tampone Anus Swab Anus

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- 1 Open the tube and place it on the tray
- 2 Take the correct swab
- 3 Insert swab approx. 3 cm into the anus
- 4 Turn several times
- 5 Insert the swab into the tube, break off the head and discard
- 6 Close the tube and place it on the tray
- 7 Wash hands with soap
- 8 Hand over the tube on the tablet

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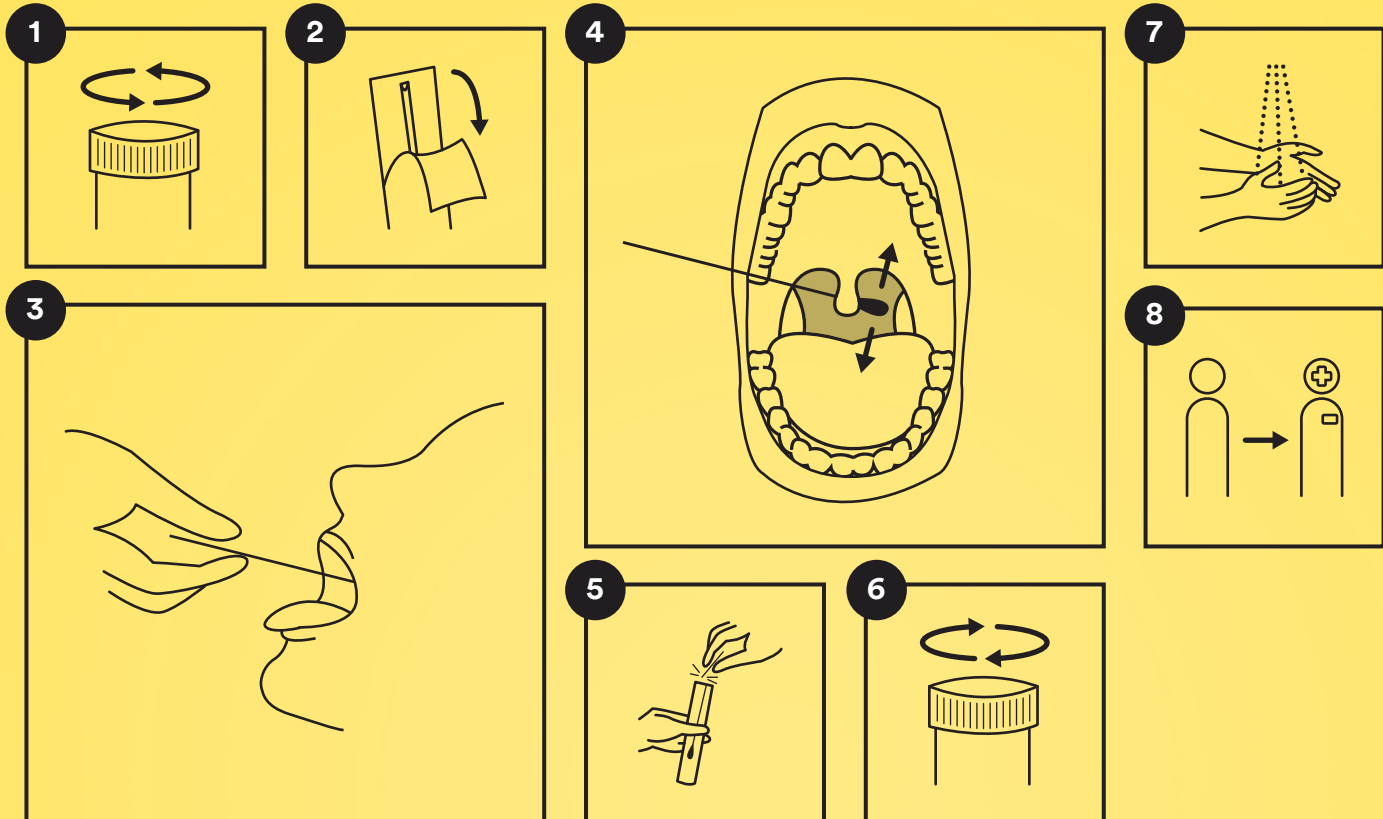
- 1 Open the tube and place it on the tray
- 2 Take the correct swab
- 3 Insert swab into the mouth
- 4 Turn several times
- 5 Insert the swab into the tube, break off the head and discard
- 6 Close the tube and place it on the tray
- 7 Wash hands with soap
- 8 Hand over the tube on the tablet

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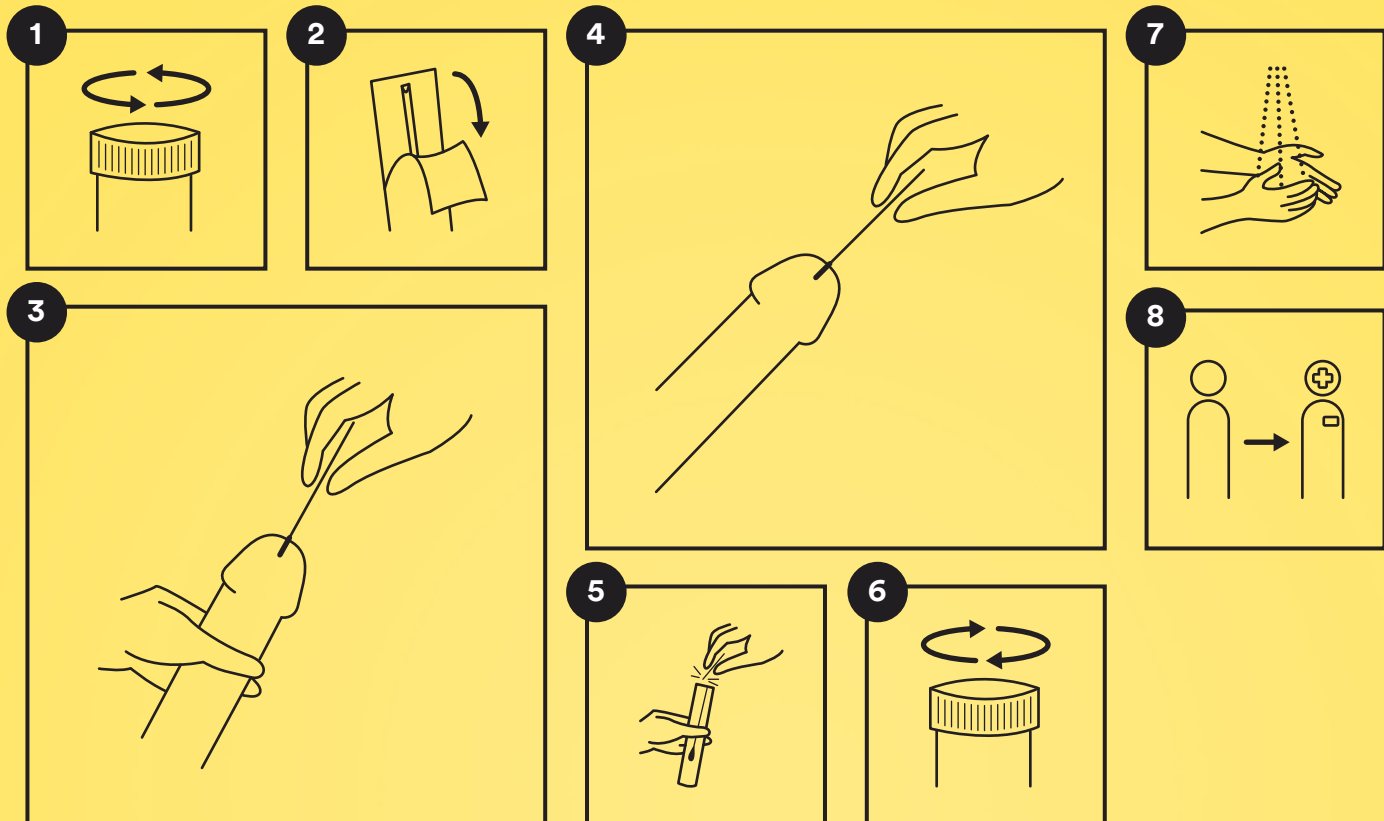
- 1 Open the tube and place it on the tray
- 2 Take the correct swab
- 3 Insert swab into the urethra
- 4 Insert once about 0.3 cm, but do not turn
- 5 Insert the swab into the tube, break off the head and discard
- 6 Close the tube and place it on the tray
- 7 Wash hands with soap
- 8 Hand over the tube on the tablet

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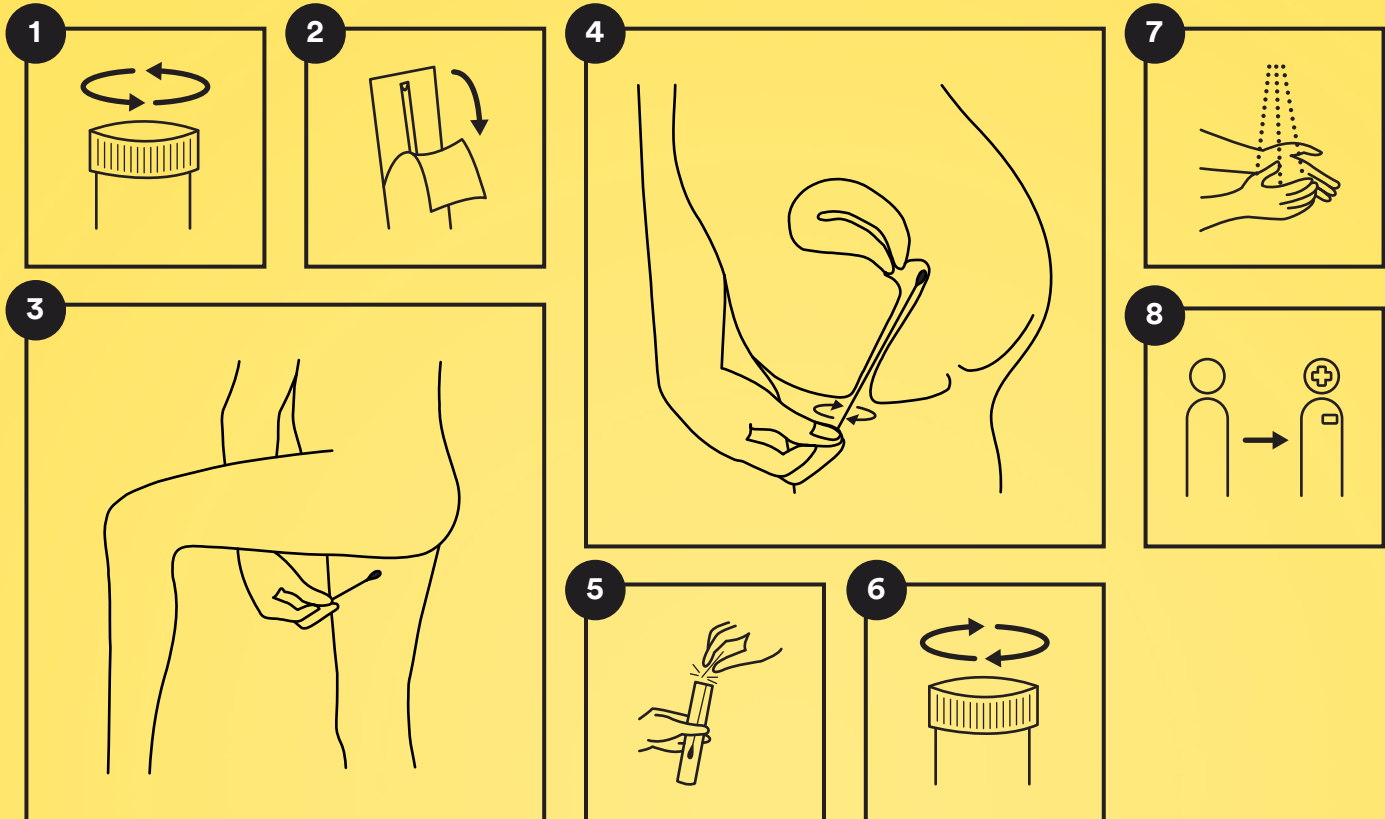
- 1 Open the tube and place it on the tray
- 2 Take the correct swab
- 3 Insert swab into the vagina
- 4 Turn several times
- 5 Insert the swab into the tube, break off the head and discard
- 6 Close the tube and place it on the tray
- 7 Wash hands with soap
- 8 Hand over the tube on the tablet

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⚠ You can't? We're here
to help you!